

Product information

FD 366 sensitive top wipes

disinfection of sensitive surfaces



FD 366 sensitive top wipes at a glance

- Practical disinfecting wipes for fast-acting disinfection of sensitive surfaces on invasive and non-invasive medical devices in hospitals, private practices, and laboratories.
- Above-average material compatibility. Particularly suitable for sensitive surfaces, such as acrylic glass, plastics, and synthetic leather coverings on treatment units.
- Exposure time in accordance with VAH only 1 minute.
- Broad spectrum of activity: bactericidal, tuberculocidal, yeasticidal, with virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV, coronaviruses as well as non-enveloped noroviruses).
- FD 366 sensitive has been tested as the active substance solution in FD 366 sensitive top wipes in accordance with current VAH methods and European standards.
- Tested in 4-field test in accordance with VAH and EN 16615.
- FD 366 sensitive top wipes are VAH-certified. FD 366 sensitive is included in the VAH list and IHO list of disinfectants.
- Wipe size: 175 x 250 mm.

Properties

FD 366 sensitive top wipes from Dürr System Hygiene are aldehyde-free, alcohol-based ready-to-use wipes for quick-acting disinfection of sensitive surfaces on invasive and non-invasive medical devices and medical furnishings in private practices, laboratories and hospitals. FD 366 sensitive top wipes are distinguished by fast efficiency and good material compatibility. Disinfected surfaces dry rapidly.

Product composition

FD 366 sensitive top wipes contain FD 366 sensitive, a fast-acting combination of alcohols and quaternary ammonium cations, as well as excipients. 100 g active substance solution contain 17 g 1-propanol, 0.125 g alkyl-benzyl-dimethylammonium chloride, excipients and water.

Antimicrobial effect

FD 366 sensitive top wipes have bactericidal, tuberculocidal, yeasticidal, virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV and coronaviruses), noroviruses. FD 366 sensitive is included in the VAH list and IHO list of disinfectants. The active substance solution has been tested under low and high load in accordance with EN 13727, EN 13624, EN 14348, EN 14476, EN 16615, as well as DVV/RKI guidelines. FD 366 sensitive top wipes are tested as ready-to-use wipes in 4-field test in accordance with VAH/EN 16615.

Directions for use

Remove FD 366 sensitive top wipes individually. Wipe the surfaces and objects to be disinfected and allow the product to take effect. Disinfect visibly contaminated surfaces with FD 366 sensitive top wipes and remove contamination. Disinfect cleaned surfaces again. Make sure that wetting is complete throughout the entire action time. Close the flow pack immediately after removing a wipe. In accordance with VAH, the action time is 1 minute (bactericidal / yeasticidal), 2 minutes for Tb bacteria. Inactivation of viruses takes place within 30 seconds to 5 minutes.

Environmental impact

The packaging consists of a composite material and can be thermally recycled. Dispose of used wipes with the domestic waste. Do not flush.

For further information see safety data sheet.

Shelf life

Do not use the product after the expiry date printed on the packaging.

After opening, 1 month.

Product information

FD 366 sensitive top wipes

disinfection of sensitive surfaces

Container sizes

Flow pack with 50 wipes
4 x 50 wipes in the box

Storage

Store in a well-ventilated place. Keep container tightly closed.

General notes

Disinfect the exterior surfaces of medical devices, such as dental curing lights, nozzles, handpieces, and contra-angle handpieces, prior to processing as required in accordance with the manufacturers' instructions. In the case of sensitive surfaces, test for compatibility in an inconspicuous place. Changes in product colour/odour may occur, especially when stored in sunlight. However, these changes in colour and/or odour have no impact on the disinfection efficacy of the product. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

Hazard warnings

FD 366 sensitive top wipes are classified and labelled according to the CLP Regulation. For further information see safety data sheet. Flash point: 33°C

Independent expert opinions – in-house investigations

The expert opinions are available upon request.

CE 0124

Application	Contact time
Recommendation for use for surface disinfection in accordance with VAH	Bactericidal / yeasticidal 1 min.

Type of activity	Tested according to	Contact time
Bacteria and yeasts		
Bactericidal / yeasticidal	EN 13727, EN 13624, EN 16615	1 min.
Tuberculocidal	EN 14348	2 min.
Viruses		
Virucidal activity against enveloped viruses (enveloped viruses including HBV, HCV, HIV as well as coronaviruses)	EN 14476	30 sec.
	DVV / RKI guideline	30 sec.
Noroviruses	EN 14476	5 min. ¹⁾

1) Testing under low burden.

Sales

Dürr Dental SE
Höpfigheimer Str. 17
74321 Bietigheim-Bissingen
Germany
www.duerrdental.com
info@duerrdental.com

Manufacturer

orochemie GmbH + Co. KG
Max-Planck-Straße 27
70806 Kornwestheim
Germany
www.orochemie.de
info@orochemie.com

