

MD 555 cleaner

Special detergent for suction systems



Foam-free special cleaner for dental suction units including drainage lines for regions with problematic tap water and drainage lines with insufficient gradient.



maintains the value of
your suction system

tested material
compatibility

recommended by leading
equipment manufacturers

intensive
cleaning effect

works perfectly in combi-
nation with Orotol plus

Contact time

 **5 %**
30-120
min

at least twice weekly

Recommendation

dissolves inorganic
deposits of
prophy powder
and **pearl products**

Function



“Descaling agent/intensive cleaning agent”:

excellent cleaning performance to tackle deposits,
including limescale resulting from very hard water,
stubborn salts, etc.



Key features

5 %

application concentration

intensive cleaning
effect to remove
deposits and
accumulations of silt

guarantees
technically and
hygienically
perfect operation of
the suction system

